

## Patient Info

Press the Patient Info or Protocol area to view or change relevant information.

## Review Data

Press **Freeze** button. Then use left and right arrows, or Go To Time button to review data. Press **Unfreeze** to return to active displays.

## Alarms, Alerts

Press the Alarms or Alerts area when lighted to learn about triggered events.

## Protocol Steps

Press any button to slide out menu. Select desired Step. Fixed protocol steps include Setup and Stop Monitoring.

## Main Display

Press to return to the selected Main Display.

## Main Menu

Press for device and measurement status and for system settings.

## EEG Config

Press to set up and check EEG connections, impedance, artifact, and video.

## Snapshot

Press to Save a screenshot of the current screen. Press to View all saved screenshots.

## Mark Event

Press to mark significant clinical events. Select an Event Category and Event Name from the drop down menu; then verify or change the event name and time. Press **OK**. Otherwise, choose Enter Event Name to type in the event.

## Change Settings

Press display area (numeric, trend, waveform). Then press **Edit Selected** to change measurement, time interval, etc.

## Time Line

Time and date display area

## View Displays

Press the button for the desired display screen. These buttons will change depending on the Step.

