



- ✓ **Stop Monitoring**
Press **Stop Monitoring** (the last of the Protocol Steps) to either pause monitoring or discharge the patient.



- ✓ **Pause Monitoring or Discharge Patient**
Press **Pause Monitoring** to suspend monitoring for a period of time (e.g. patient transport, etc) with the intent to resume monitoring.
- OR -
Press **Discharge Patient** to discontinue monitoring and discharge a patient. Confirm that you want to proceed with discharging the patient.

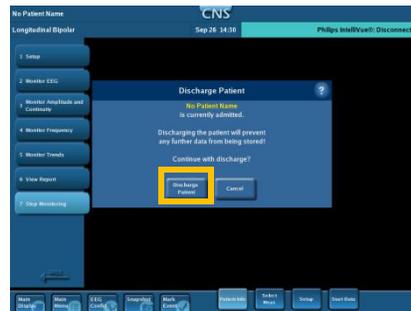

- ✓ **Protocol Modified**
This menu only appears if any protocol changes have been made. Press **Discard Changes** or contact your supervisor to determine if changes should be saved.

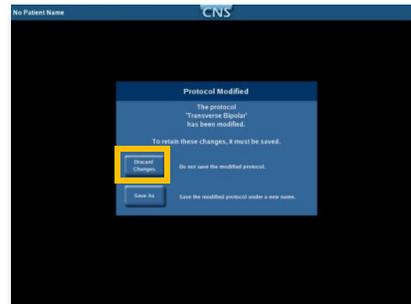

- ✓ **Archive Now, Archive Later, or Erase**
Archiving lets you save the patient data file to a USB Thumb Drive or the hospital network. Automatic archiving to the network may be standard practice in some hospitals. If unsure, choose **Archive Later** to store the patient file on the CNS Monitor.

*Do not **Erase** data unless directed!*

- ✓ **Power Down the Monitor**
Press the **ON/OFF Button** located on the lower right corner of the monitor. The lighted ON/OFF button will go dark once the power down sequence is complete. Once the light goes out, it is safe to unplug the CNS Monitor from the electrical outlet.

